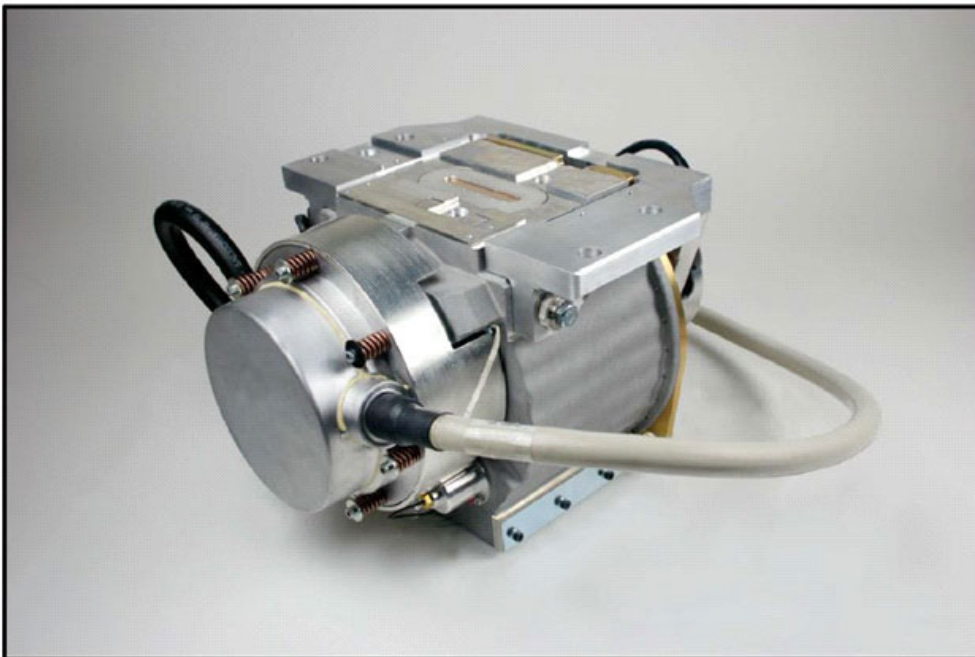
X-Ray Tube Housing Assembly

Gaine radiogène
Röntgenröhrengehäuse
Coraza del tubo de rayos X
X射线管组件

AU240

453575083895 Rev.D

TECHNICAL INFORMATION

Section 1.0:

Symbol Legend, Symbole légende, Symbol Legende, Leyenda del símbolo, 符號傳奇 p. 3-4

Section 2.0:

Technical Information, Informations techniques, Technische Informationen, Información técnica, 技术资讯 p. 5-15

Section 2.1:

Product Specifications, Caractéristiques techniques du produit, Produktspezifikationen, Especificaciones del producto, 产品规格 p. 5-10

Section 2.2:

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Intended Use

This X-ray Tube Housing Assembly is design is to be operated as part of a diagnostic imaging system. The X-ray imaging system may only be operated by persons who have the necessary training and expertise in radiation protection and who have been instructed in how to operate the X-ray system. General Information regarding Installation, Maintenance, Transportation, Safety, Disposal, and Return are stated in the General Information document supplied with this device or on-line at chronosimaging.com.

Copyright

© 2023 Chronos Imaging, LLC

All rights reserved. Reproduction in whole or in part is prohibited without the prior written consent of the copyright-holder. Chronos Imaging, LLC reserves the right to make changes in specifications or to discontinue any product, at any time without notice or obligation, and is not liable for any consequences resulting from the use of this publication.

Equipment is subject to change without notice. All changes will be in compliance with regulations governing manufacture of medical equipment. Printed in the USA. Document originally drafted in English.

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This tube is a monopolar anode grounded X-Ray tube assembly. The cathode (-) high voltage cable is connected to the X-Ray Tube Assembly via a ceramic insulator while the "free" end is fitted with a Federal Standard type connector with 3 conductors.

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Equipment not suitable for use in the presence of flammable anesthetic mixture with air or with oxygen or nitrous oxide.

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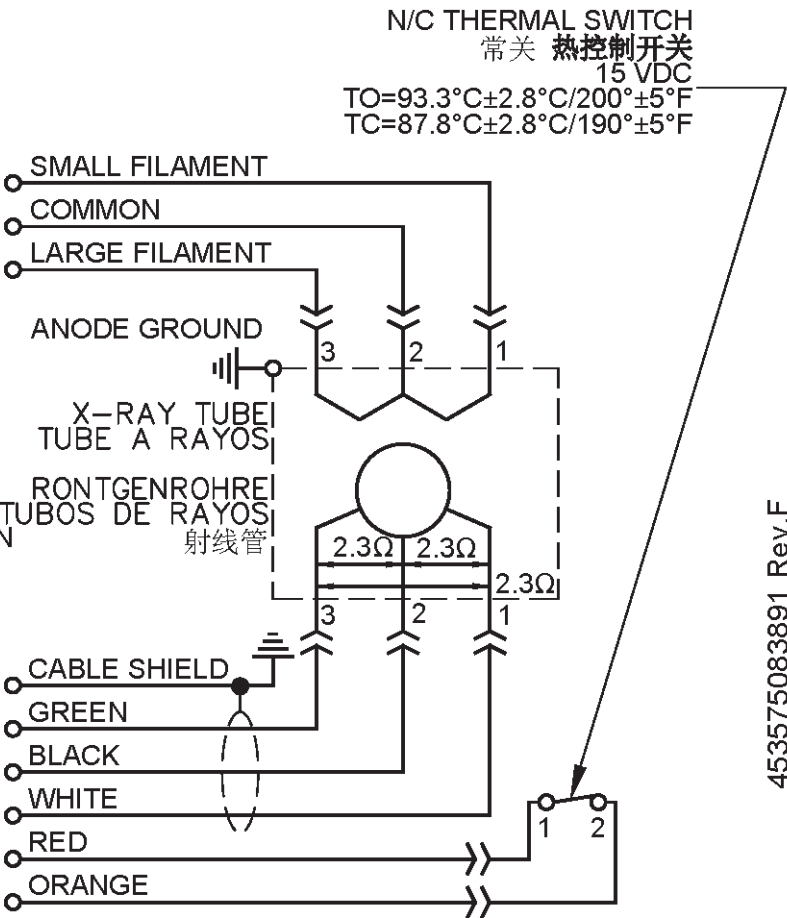
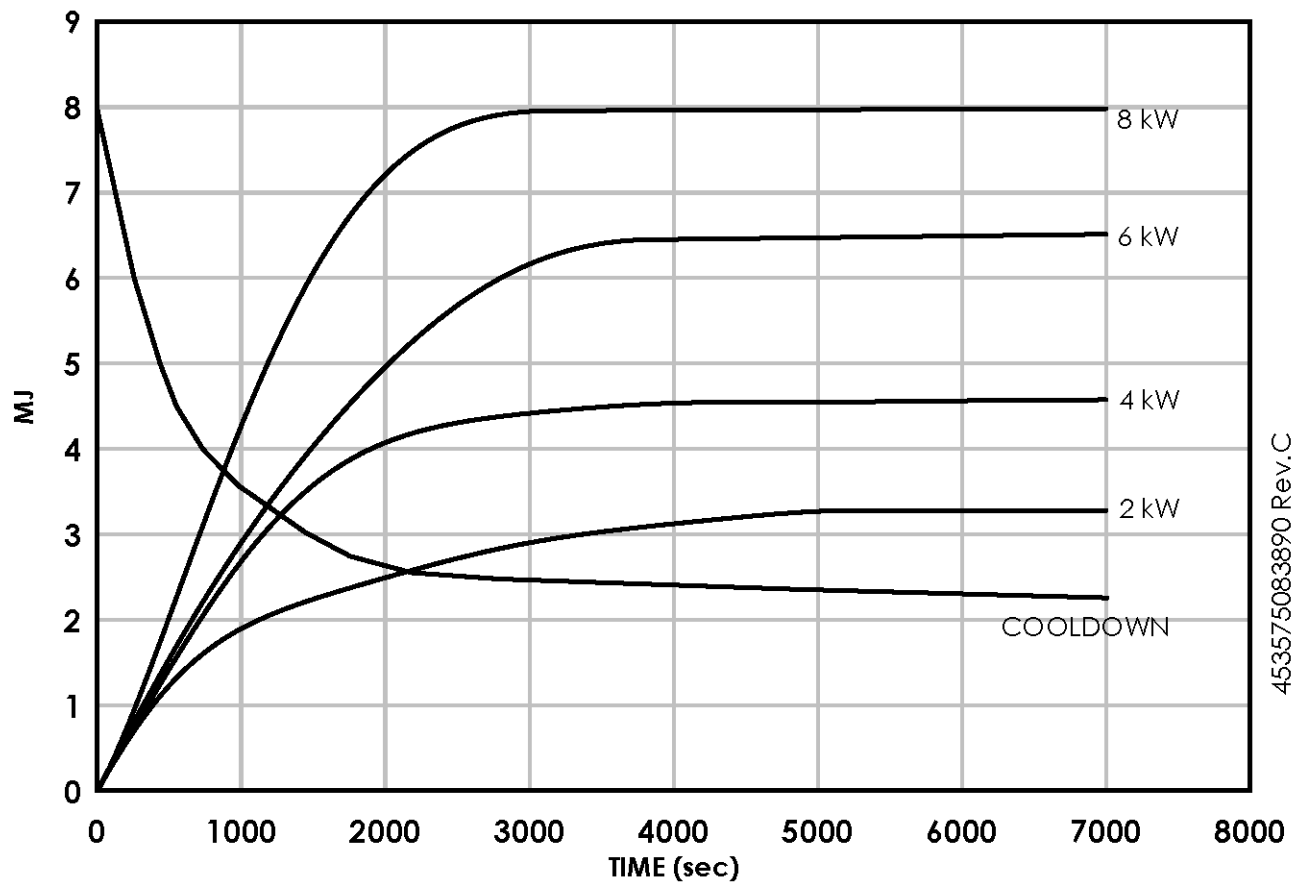
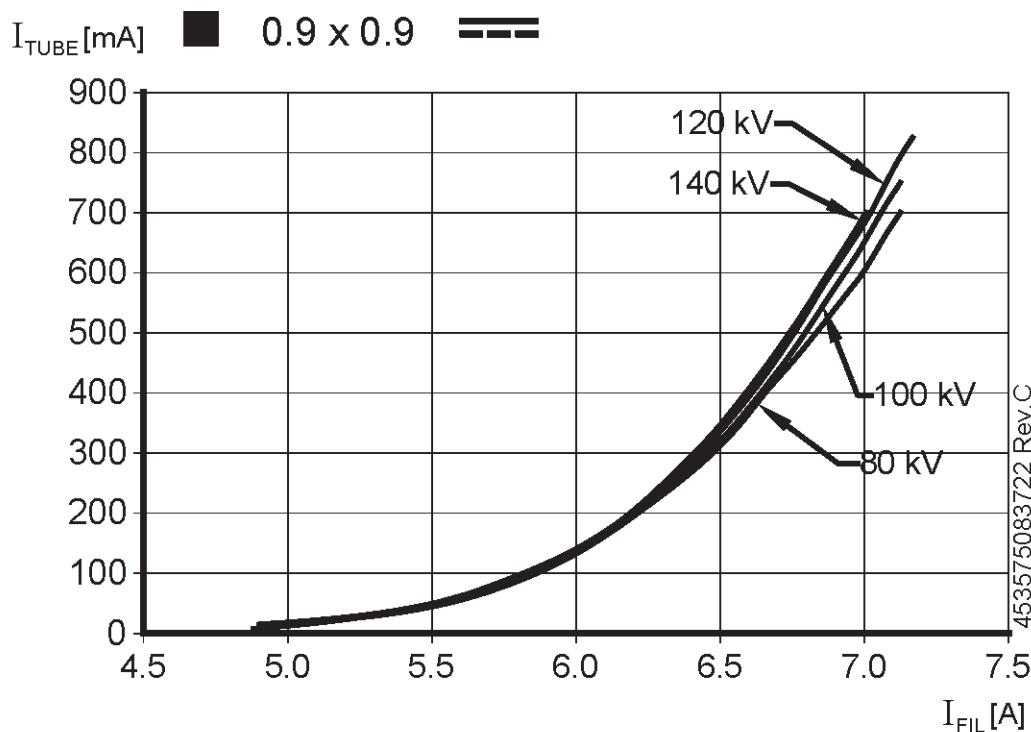
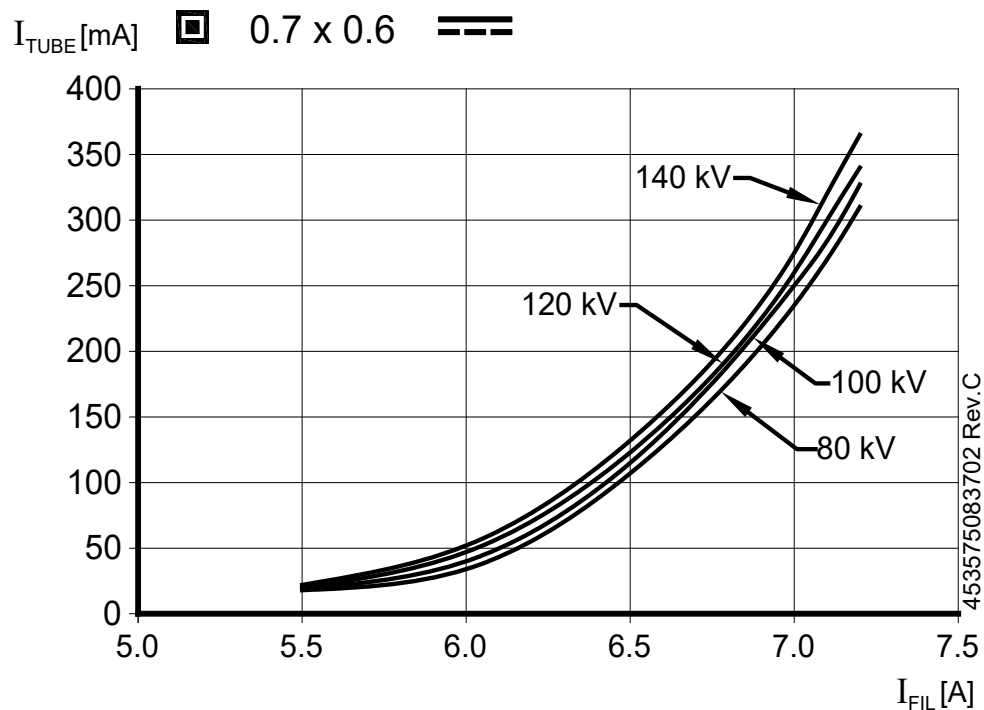
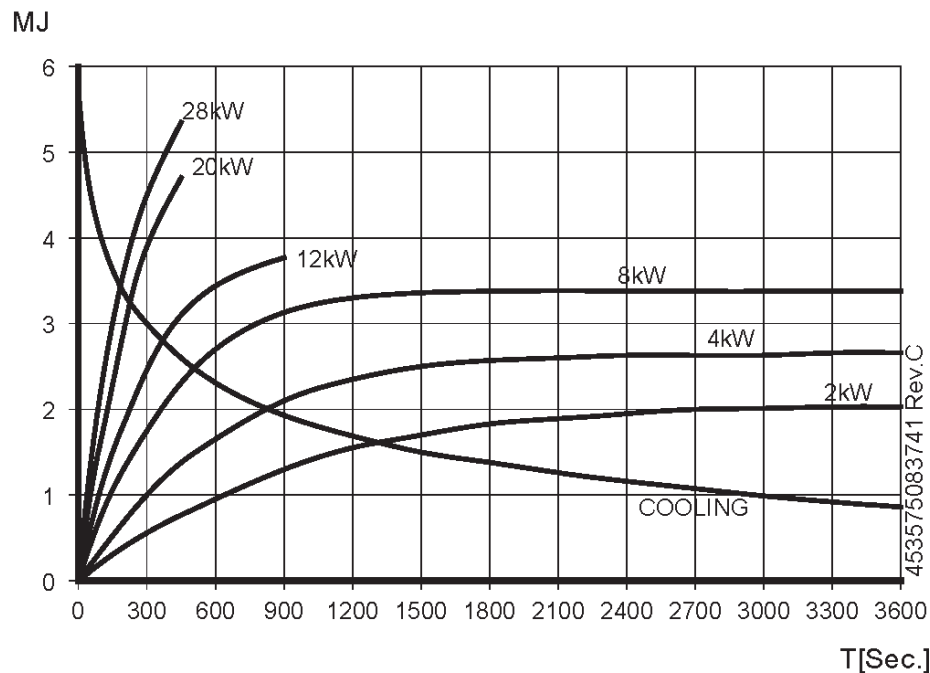
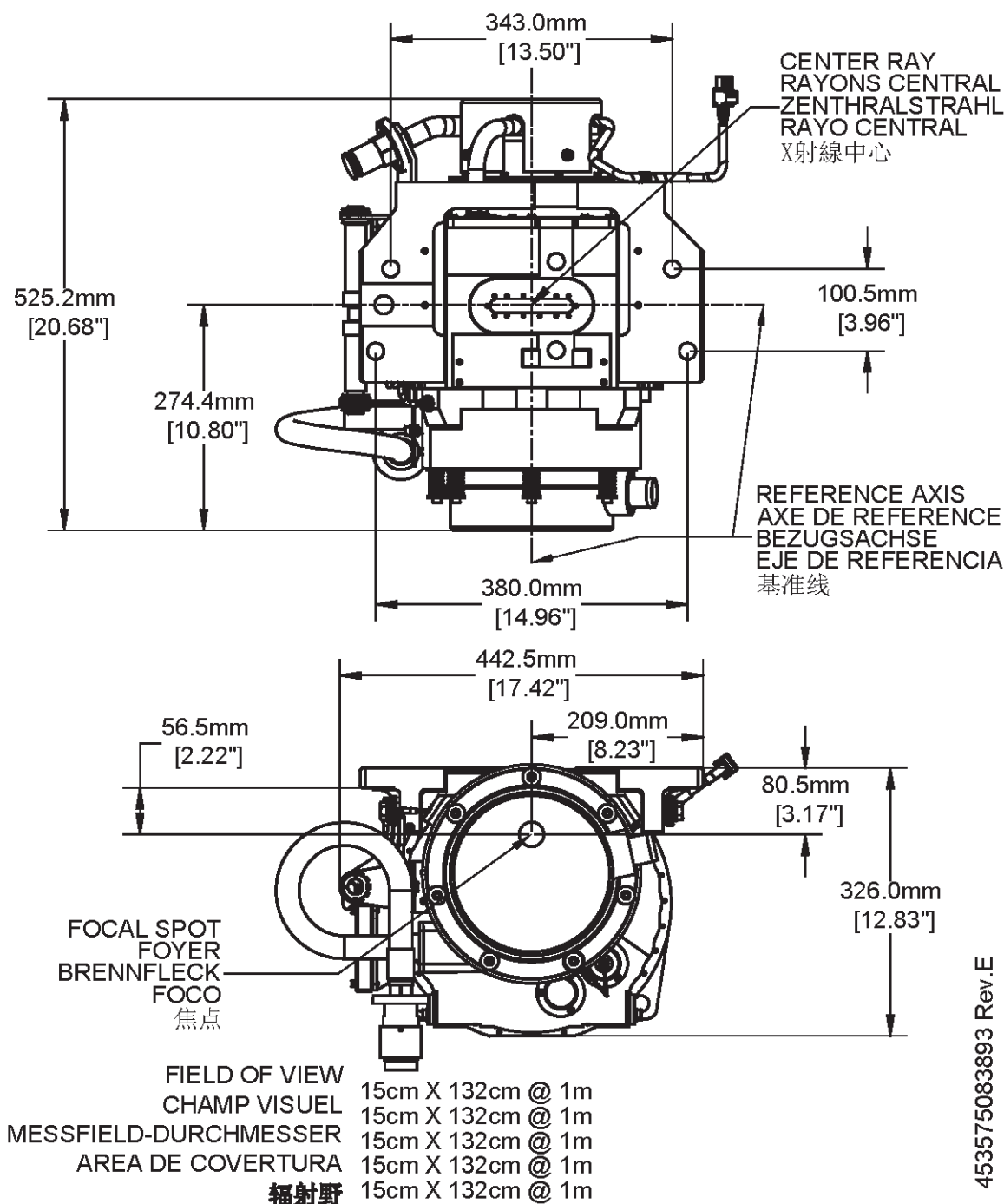
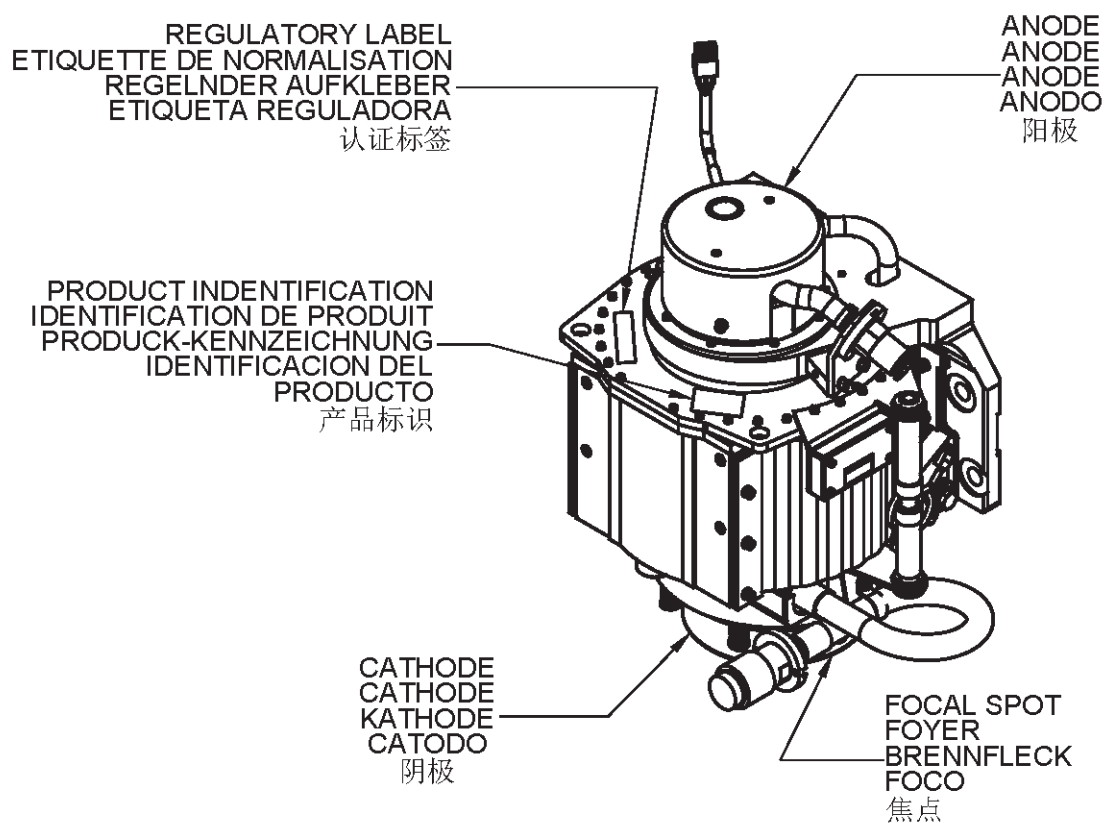
Device Manufacturer

Chronos Imaging Facility
555 North Commerce Street
Aurora, Illinois 60504 USA

European Union Authorized Representative

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MDSS GmbH